#### **Hersteller**

**lege artis** Pharma GmbH + Co. KG  
Postfach 60, D-72132 Dettenhausen  
Breitwasenring 1, D-72135 Dettenhausen  
Telefon +49 (0) 71 57 / 56 45 - 0  
Fax +49 (0) 71 57 / 56 45 50  
E-Mail: [info@legeartis.de](mailto:info@legeartis.de)  
[www.legeartis.de](http://www.legeartis.de)

#### **Bezeichnung**

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**Name**

ORBAT sensitive  
Liquid for gingival retraction

**Composition**

Aluminium sulfate solution 25%

**Indications**

For retraction of the gingiva.  
For tissue management during direct and indirect restorations.  
For haemostasis of gingival bleeding and for stopping the exudation before impression taking.

**Contra indications**

Do not use on patients with allergy to aluminium compounds.

**Side effects**

Allergic reactions to aluminium compounds are possible. Aluminium sulfate can lead to slight desquamation of the mucosal epithelium.  
In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

**Notices and precautions**

Do not use with impregnated retraction cords.  
If heavier capillary bleeding needs to be handled, ORBAT forte should be used for gingival retraction and to stop gingival bleeding.  
Avoid swallowing. After swallowing, rinse mouth. Drink plenty of water in small sips (dilution effect). If you feel unwell, seek medical advice / attention.  
In case of skin contact, wash gently with plenty of water and soap. If skin irritation occurs, seek medical advice / attention.  
If eye contact occurs, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.  
The safety data sheet can be downloaded from [www.legeartis.de](http://www.legeartis.de) or requested by E-Mail from [sicherheitsdatenblaetter@legeartis.de](mailto:sicherheitsdatenblaetter@legeartis.de).

**Mode of application**

Retraction of the gingiva

For gingival retraction before impression taking, soak a non-impregnated retraction cord (ring) with ORBAT sensitive. Remove the cord after approx. 1 - 3 minutes (or even longer) and gently rinse if necessary.

Tissue management during direct and indirect restorations

For gingival retraction and to stop exudation, lay a ORBAT sensitive soaked retraction cord (ring) into the sulcus or use a suitable applicator to rub in ORBAT sensitive directly under light pressure.

Before cementation or adhesive attachment of the restoration, care must be taken that any gingival bleeding and exudation are stopped and excess aluminium sulfate has been removed thoroughly with plenty of water spray.

Exudation and gingival bleeding

Swab away the exudate or blood and apply ORBAT sensitive with soaked cotton pellet or another suitable applicator under light pressure.

**Further notices**

Aluminium sulfate has a good astringent effect. It causes local haemostasis by contracting the upper layer of the gingiva.

ORBAT sensitive can be applied directly from the dropper bottle to the applicator or to suitable retraction cords. However, it is better to add a few drops from the bottle into a dappen dish or something similar in order to soak the cords there well or to take the solution with the applicator from there.

Keep out of reach of children.

Only for professional dental use.

Do not pour back residues of the solution into the bottle.

**Shelf life**

ORBAT sensitive should not be used after the expiry date.

**Storage instructions**

Close the bottle immediately after use.

**Dosage form and package size**

20 ml solution in PE dropper bottle Item number 0032327

**Date of revision**

2021-10

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"  
Shows Item number of the manufacturer

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