(DE) Fachinformation

REMOT

Hersteller

lege artis Pharma GmbH + Co. KG
Postfach 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telefon +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

Bezeichnung

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Name

REMOT prophylaxis paste
For removal of dental plaque, prophylaxis of caries and final polish.
For dental use.

Composition

Silica, macrogols, zirconium(IV)-oxide, sorbitol, glycerol, peppermint oil, chlorhexidine digluconate and cocamidopropyl betaine.

Indications

REMOT prophylaxis paste for professional tooth cleaning. REMOT removes plaque and discolorations (e.g. from tea, coffee or tobacco) from teeth and therefore prevents caries and periodontal disease. Also suitable for final polish.

Contra indications / Side effects

Treatment is not recommended in the case of known allergies to one of the ingredients of REMOT.

In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

Notices and precautions

The well-balanced combination of abrasive materials effects a careful removal of the dental plaque from teeth without damaging the enamel.*

Do not use REMOT without respective protective measures if control over the patient's swallowing reflex is not guaranteed.

Avoid swallowing. After swallowing, rinse mouth. Get medical advice / attention if you feel unwell.

Avoid skin contact. In case of skin contact, wash gently with plenty of water and soap.

Contact with the eyes must be avoided. After accidental eye contact, rinse gently with water for a few minutes, remove any existing contact lenses if possible. Continue rinsing. Seek medical advice / medical attention immediately.

The safety data sheet can be downloaded from www.legeartis.de or requested by E-Mail from sicherheitsdatenblaetter@legeartis.de.

Mode of application

Before treatment, the patient must be prepared accordingly, i. a. protecting the eyes, skin, clothing and lips from splashes or mechanical irritation, and possibly putting the patient's glasses aside. After removal of the tartar / coarse impurities and possible previous treatment with a product with higher abrasiveness (in the case of heavier deposits), REMOT is respectively withdraw hygienically with suitable instruments or will be

* Stookey/Schemehorn method with pumice as reference substance for prophylaxis pastes

Literatur: Stookey GK, Schemehorn BR: A Method for Assessing the relative Abrasion of Prophylaxis. Materials J Dent Res 1979. 58:588.

RDA values obtained: < 7 for "REMOT implant" and 7 for product "P" fine abrasion grade (pink)

For comparison:

Hefferen method with calcium pyrophosphate as reference substance for toothpastes

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RDA values obtained: 66 for "REMOT implant", 71 for product "P" fine abrasion grade (pink) and 106 for REMOT

placed in a dappen dish or a comparable container, so that the paste can be used from there during the treatment. REMOT will be applied on teeth with suitable rubber cups and prophylaxis brushes. The treatment is to be repeated with respectively fresh paste until then the entire surface is cleaned / polished. The saliva should be aspirated during treatment to avoid swallowing the paste. At the end of the treatment, allow the patient's mouth to be thoroughly rinsed and carefully clean the patient's face from splashed without getting the product into the eyes.

Further notices

REMOT has a pleasantly fresh taste, cleans the teeth and gives them shine.

Keep out of reach of children.

Do not pour back residues of the paste into the container.

Shelf life

REMOT shall not be used after the expiry date.

Storage instructions

Close the REMOT container after use and always keep it closed.

Dosage form and package size

75 g prophylaxis paste Item number 0032317

Date of revision

2023-11

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"
Shows Item number of the manufacturer

Symbol of "Medical Device"

Symbol of "Unique device identification"

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