(DE) Fachinformation

**FOKALDRY**

CE 0124

**Hersteller**

**lege artis** Pharma GmbH + Co. KG  
Postfach 60, D-72132 Dettenhausen  
Breitwasenring 1, D-72135 Dettenhausen  
Telefon +49 (0) 71 57 / 56 45 - 0  
Fax +49 (0) 71 57 / 56 45 50  
E-Mail: [info@legeartis.de](mailto:info@legeartis.de)  
[www.legeartis.de](http://www.legeartis.de)

**Bezeichnung**

FOKALDRY

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### Name

FOKALDRY

Cleaning solution for the cavities, tooth stumps and root canals  
For dental use only

### Composition

Diisopropylether, petroleum ether, Acetone and isopropyl alcohol

### Indications

Cleaning, degreasing and drying of cavities, tooth stumps, root canals, crowns bridges and brackets.  
Removal of plaque on the hard substance of the tooth.  
Removal of uncured cement and composite remnants.

### Contra-indications

Allergy to isopropyl alcohol.

### Side effects

In case of gingival or mucous membrane contact, burning may occur, avoid contact as much as possible. Inhaling FOKALDRY may cause irritation of the throat and pharynx. When inhaling larger quantities, headache, nausea, and sensation of dizziness may occur.  
In rare cases, repeated contact may lead to an allergic reaction to isopropyl alcohol.  
In the event of serious incidents in connection with the product, please contact the manufacturer and the responsible authority immediately.

### Notices and precautions

Avoid swallowing. After swallowing, rinse mouth. Do not induce vomiting. In case of vomiting danger of aspiration. Seek medical advice/medical attention immediately.  
In case of skin contact, wash with plenty of water and soap. If eye contact occurs, rinse with water for a few minutes, remove any existing contact lenses if possible.  
The safety data sheet can be downloaded from [www.legeartis.de](http://www.legeartis.de) or requested by E-Mail from [sicherheitsdatenblaetter@legeartis.de](mailto:sicherheitsdatenblaetter@legeartis.de).

### Mode of application

A cotton wool pellet of appropriate size is soaked with FOKALDRY and used to clean the cavity resp. the stump.  
Crowns, bridges and brackets should be cleaned with FOKALDRY prior their permanent fixation. In case of plastic crowns, partial crowns or brackets embrittlements or matting during application may occur depending on the used plastic. In this regard, the information from the respectively manufacturer regarding material compatibility has also to be considered. Moisten the root canal with FOKALDRY, and dry it with paper points. After evaporation FOKALDRY does not leave a smear film.  
Take care that the liquid does not come over the apex into the periapical tissue, while cleaning the root canal.  
As far as possible a rubber dam is to be used.

### Further notices

FOKALDRY serves to remove fat-containing remainders of tooth preparations and humidity to achieve a better adaption resp. compound with the rebasing resp. filling material and anchorage cements. Too extensive use of FOKALDRY may cause dehydration of dentin and upset the physical balance of interstitial liquid of the pulp.

Do not pour back residues of the solution into the bottle.

Keep out of reach of children.

### Shelf life

FOKALDRY shall not be used after the expiry date.

### Storage and preservation

Close the bottle immediately after use and keep protected from light and heat. FOKALDRY is an easily inflammable liquid, therefore, do not handle or store near sources of ignition. In the case of larger quantities of evaporated liquid, ensure sufficient aeration.

### Administrative form and package sizes

80 ml solution Item number 0032305

500 ml solution (480-ml reserve bottle with additional spout for decanting and 20 ml application bottle with 3 drops dispensers for easy dropwise removal) Item number 0032306

### Date of revision

2023-09

Symbol of "Expiry date"

Symbol of "Batch number"

Symbol of "Pay attention to the directions for use"

Symbol of "Manufacturer"

Symbol of "Catalogue number"  
Shows Item number of the manufacturer

Symbol of "Medical Device"

Symbol of "Unique device identification"

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